**MTA-FILLAPEX**

# PORTUGUÊS

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Bioceramic root canal sealer

## INTRODUCTION

MTA-FILLAPEX is a bioceramic root canal sealer based on MTA (mineral trioxide aggregate). Its composition provides the following qualities:

- Indicated for the permanent sealing of root canals;
- High radiopacity;
- Low expansion during setting;
- Low solubility in contact with tissue fluids;
- Optimal flow characteristics for insertion;
- Does not stain the tooth;
- Eugenol-free material: does not interfere with the setting of resin cements.

CAUTION: Federal law restricts this device to sale by or on the order of a dentist.

## COMPOSITION

Base Paste: Salicylate Resin, Natural Resin, Calcium Tungstate, Nanoparticulated Silica, Pigments;

Catalyst Paste: Diluting Resin, Mineral Trioxide Aggregate, Nanoparticulated Silica, Pigments.

## INDICATION

MTA-FILLAPEX is a root canal sealer intended for the permanent sealing of root canals and may be used in combination with root canal obturation materials.

## CONTRAINDICATION

In patients with hypersensitivity against the resins or other components of the product.

## WARNINGS

- MTA-FILLAPEX contains resins which may sensitize susceptible individuals. Do not use it in patients allergic to the resins or other components of the product;
- Avoid contact with eyes or skin. In case of contact, rinse immediately with water;
- Avoid contact with oral mucosa. In case of contact, rinse with water and prevent swallowing of product. In case any sensitivity persists, seek medical attention promptly;
- If the syringe becomes contaminated with saliva or blood during application, dispose of the syringe and do not use on an additional patient.
- Ensure that the lids of the base paste and catalyst are not switched, because switching them can cause

hardening of the product inside the tube.

## **PRECAUTIONS**

During the use of this product, it is recommended the patient and professional use suitable protective clothing, eye protection and gloves.

## **STORAGE**

- Keep in dry and cool place, away from humidity and sources of heat;
- Do not store in the refrigerator;
- Keep the product tightly capped.

## **TECHNIQUE OF USE**

1. Root canal preparation: Prior to insertion of MTA-FILLAPEX, the root canal should be prepared and cleaned according to the selected endodontic technique. The device requires an isolated dry field; therefore, moisture found in the canal or in the ambient humidity is sufficient to initiate MTA-FILLAPEX hardening;
2. Mixing:
  - Dual syringe: The dual syringe ensures equal mixing in a 1:1 ratio. Press plunger to extrude material directly onto a glass slab, mixing pad or into the tooth canal. The cement should be used immediately after mixing;
  - Tubes: Mix equal volume units (1:1). Mix for 30 seconds to a homogeneous consistency;
3. Insertion: Use MTA-FILLAPEX with gutta-percha or silver points, coating them with a thin layer of the cement and filling the root canal according to the selected technique. MTA-FILLAPEX can be applied in the root canal with a Lentulo spiral or, directly, with an applicator tip adapted to the self-mixing tip;  
**WARNING:** The self-mixing tip must be discarded after use.
4. Removal of the root canal filling: Use the conventional techniques for the removal of gutta-percha fillings;
5. Working time: 23 minutes.

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