## Instructions for use

1005102016

### Choose your language

- |                       |                       |
|-----------------------|-----------------------|
| <b>PT</b> Português   | <b>SK</b> Slovensky   |
| <b>EN</b> English     | <b>PL</b> Polski      |
| <b>ES</b> Español     | <b>ET</b> Eesti       |
| <b>FR</b> Français    | <b>LT</b> Lietuviškai |
| <b>IT</b> Italiano    | <b>LV</b> Latviski    |
| <b>RO</b> Română      | <b>CS</b> Český       |
| <b>NL</b> Nederlands  | <b>GA</b> Gaeilge     |
| <b>DE</b> Deutsch     | <b>HU</b> Magyar      |
| <b>SV</b> Svenska     | <b>TR</b> Türkçe      |
| <b>DA</b> Danish      | <b>MT</b> Malti       |
| <b>FI</b> Suomi       | <b>BG</b> Български   |
| <b>HR</b> Hrvatski    | <b>RU</b> Русский     |
| <b>SL</b> Slovenščina | <b>EL</b> Ελληνικά    |

# Português

## INDICAÇÕES

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# English

## INDICATIONS

For the support of restorations and prosthetic crowns.

## COMPOSITION

Peeso drills: Stainless steel

Rubber Stops: Latex

Template for radiographic measurement: Plastic

REFORPOST® GLASS FIBER: Glass fiber (80%); pigmented resin (19%), stainless steel filament (1%).

REFORPOST® CARBON FIBER: Carbon fiber (72%); pigmented resin (22%), stainless steel filament (6%).

## TECHNIQUE OF USE

Select the pin using the template and the periapical x-ray of the tooth;

Remove some of the root canal filling with heated instruments to the predetermined depth. Allow at least 3 to 5 mm of filling at the root apex;

Prepare the canal with the drill corresponding to the diameter of the post.

Place the post and cut at the desired limit with a diamond drill at high speed under refrigeration;

Clean the post with alcohol and dry it;

Apply a layer of silane (SILANO ANGELUS®) and let it dry for 1 minute;

**WARNING:** It is not necessary to apply the silane on REFORPOST® CARBON FIBER.

Apply the ANGELUS® FUSION-DURALINK® CATALYST to the post according to its instructions.

Clean the canal with the air-water spray and dry it with air and absorbent paper points;

Etch the canal with ANGELUS® 37% PHOSPHORIC ACID for 15 seconds;

Rinse and dry with air and absorbent paper points. Apply ANGELUS® FUSION-DURALINK® PRIMER in the canal and wait for 1 minute;

Apply ANGELUS® FUSION-DURALINK® CATALYST according to the instructions. This adhesive requires no curing;

Spatulate the resinous cement selected\* and fill the conduit;

Put the post in the canal, remove excess of the cement and wait for the polymerization.

Auto-adhesive cements eliminate the needs for the steps of acid conditioning and adhesive application.

Perform conditioning and apply an adhesive on the remaining crown and on the post. Build the crown core with composite resin, or with ANGELUS® REFORCORE®. REFORCORE® is a prefabricated glass fiber core.

#### USE OF ACCESSORY POSTS IN WEAKENED ROOTS AND WIDE CANALS

Fill the remaining conduit spaces with accessory posts REFORPIN ANGELUS to reduce the thickness of the cement and better imbrication.

#### WARNINGS | PRECAUTIONS

The use of fiber posts is not suitable for extensive rehabilitation and for use in patients with parafunctional habits;

Prefabricated fiber posts should be used only on teeth with a minimum of 2 mm in dentin height;

Fiber posts should only be cut with water refrigerated high speed diamond burs. Do not use scissors or cutting pliers. Use goggles and a mask to prevent inhalation of particles generated during cutting;

Fiber posts must be completely covered by the cementing agent or the composite resin.

**IMPORTANT:** The indication of any post depends on careful evaluation of the quality of endodontic treatment and periapical health.

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