CALCADOR DE BERGER

PORTUGUÊS

DEFINIÇÃO

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ENGLISH

DEFINITION

Plugger for condensation of gutta-percha in filling the root canal.

INDICATION

Condensation and cut of gutta-percha used in filling the canal.

COMPOSITION

Handle: polyolefin polymers.

Active part: stainless steel.

TECHNICAL INFORMATION

The active part is presented in 3 lengths with 3 tip diameters:

- No. 01: 23 mm / 0,5 mm
- No. 02: 21 mm / 0,8 mm
- No. 03: 19 mm / 1,0 mm

USE TECHNIQUE

1. Insert the gutta percha cone to the work length.
2. Repass the patency instrument 1 mm beyond the foramen with a few filing movements.
3. Handle the cement and insert it with memory instrument, brushing the canal walls.
4. Reinsert the gutta percha cone, as apical as possible.
5. Cut the cone in the middle third with Berger's condenser well heated in a single movement.
6. Condense the cone with Berger's Condenser for 20 seconds with oscillating movement and pull it.
7. Use the Mc Spadden Condenser Gutta diameter 40 in the filling for 5 seconds, at 5 mm below the radiographic vertex.
8. Condense the filling with Berger's Condenser.
9. X-ray and if necessary repeat the procedure.
10. Fill the medical and cervical third with the Lateral Condensation Technique.

WARNINGS | PRECAUTIONS

- Evaluate the condition of the product after each use.
- Use in the sequence defined in the use technique.
- Sterilize the product before each use.
- Berger's Condenser must be sterilized in autoclave at 121 degrees Celsius for at least 15 minutes, or at 134 degrees Celsius for at least 3 minutes, in accordance with EN ISO 17665-1.

STORAGE

After sterilization store in a dry place which ensures the maintenance of the sterilization.

① **ATTENTION:** The information contained in these instructions is based upon scientific, clinical and laboratory studies. Nevertheless, the success of the procedures depends upon the accurate diagnosis and correct indication as well as careful operating technique.

* Product of exclusive use of the Dental surgeon.

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