ANGIE ASSENTO ODONTOPEDIÁTRICO

PORTUGUÊS

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ENGLISH

INSTRUCTIONS FOR USE

1. Unroll the PEDIATRIC DENTAL SEAT on a flat surface, and

ensure that the area is free of objects which can tear or pierce the product;

2. To inflate: Remove the cover of the air inlet valve (fig. 01); and with the three-way syringe of air/water of the dental clinic, start the filling process (fig. 02) until the PEDIATRIC DENTAL SEAT attains the pressure required (the wrinkles disappear); insert the valve rubber cover again; repeat the procedure for the headrest and knee support;
3. The deflate: Remove the cover of the valve withdrawing all the air. **Warning:** Maintain the seat far away from fire and hot objects and do not overdo the pressure, as it can burst the seals (fill until the wrinkles disappear).

Note: For both inflating and deflating, press the valve base with the fingers, as there is a safety membrane. Avoid excessive pressure. Do not try to remove the last folds from the inflatable, as it may damage the part.

CLEANING AND STORAGE

1. If the PEDIATRIC DENTAL SEAT is wet, it is advisable to let it dry before preparing it for storage;
2. For cleaning, use a clean and soft cloth and alcohol 70% for disinfection.
3. Do not use detergents or other chemical products;
4. Before storing the product ensure that it is totally without air, and store it in a dry well-ventilated place;
5. Do not place heavy or sharp objects near or on top of the mattress to avoid damaging and cutting it.

HOW TO POSITION IT IN THE DENTAL CHAIR

- With the PEDIATRIC DENTAL SEAT inflated, place it on the dental chair and use the finest strips to attach it. **Warning:** The arms of the PEDIATRIC DENTAL SEAT are for containing children, if the dentist so requires, and they are not inflated.

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