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PORTUGUÊS

DEFINIÇÃO

Clareador caseiro 10 ou 16%.

INDICAÇÃO

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10 or 16% home bleaching gel.

INDICATION

Dental whitening at home under the direction of the dentist.

COMPOSITION

- 10% carbamide peroxide (3.6% Hydrogen peroxide) or 16% carbamide peroxide (5.8% Hydrogen peroxide);
- Thickener;
- Desensitizing agents;
- Neutralizing agent;

- Preservatives;
- Glycol;
- Flavouring substances;
- Deionized water.

TECHNIQUE OF USE

1. Dispense 1 drop of gel in the whitening tray in the center of the tooth to be whiten. In case of excesso gel, remove it with a toothbrush;
2. Insert the tray with the bleaching gel in the mouth;
3. The application time should be a minimum of 2 hours/day;
4. After use, remove the tray, rinse, dry and store in a dry place away from moisture and heat;
5. Brush teeth slowly. Patients with a history of tooth abrasion or wear should wait for 30 minutes before brushing;
6. The average whitening period is 2 - 3 weeks depending on time of use and initial tooth shade.

STORAGE

- Store the product at temperatures between 5 and 25°C. Do not freeze the product. Protect it from direct light and heat.

PRECAUTIONS

- Tooth whitening should always be carried out under the supervision of a dentist;
- The patient must be in good oral and general health;
- This product is not recommended during pregnancy, breastfeeding and for children under 18;
- The patient should be instructed on the technique and treatment results;
- The patient should be warned that CLÀRIANT ANGELUS® HOME gel does not whiten direct or indirect restorations;
- Tooth bleaching may be ineffective in cases of structurally altered teeth (amelogenesis, dentinogenesis imperfecta and fluorosis stains) or very pronounced discoloration caused by tetracycline and other drugs taken systemically during tooth formation;
- The patient should be warned of a possible increase in tooth sensitivity to cold stimuli during treatment. To stop this sensitivity, simply decrease the frequency (e.g., whiten every other day) or time of use. The use of the desensitizing gel CLÀRIANT ANGELUS® D-SENSE applied to the tray instead of the bleaching gel is also recommended;
- This product cannot be used on patients allergic to any of its components.

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