BIO-C SEALER

CHOOSE YOUR LANGUAGE

ENGLISH.....	3	SLOVENSKY	48
ESPAÑOL.....	6	POLSKI	51
PORTUGUÊS	10	EESTI	55
FRANÇAIS	13	LIETUVIŠKAI	58
ITALIANO	17	LATVISKI	62
ROMÂNĂ	20	ČESKY	65
NEDERLANDS.....	23	GAEILGE.....	68
DEUTSCH	27	MAGYAR	72
SVENSKA	31	TÜRKÇE	75
DANISH.....	34	MALTI	79
SUOMI	37	БЪЛГАРСКИ	82
HRVATSKI	41	РУССКИЙ	86
SLOVENŠČINA	44	ΕΛΛΗΝΙΚΑ	90

ENGLISH

DEFINITION

Injectable endodontic bioceramic sealer ready for use.

INDICATIONS

1. Sealing the root canal of permanent teeth;
2. Internal reabsorption treatment.

COMPOSITION

Calcium Silicates, calcium aluminate, calcium oxide, zirconium oxide, iron oxide, silicon dioxide and dispersing agent.

TECHNICAL INFORMATION

- Biocompatible product ready for use;
 - Working time: 60 minutes;
 - Setting time: 120 minutes (2 hours) after insertion into the canal, according to ISO 6876;
- Important: Dry with paper cones, but without excessively drying the canal; the moisture from the dentin tubules will initiate the material's setting reaction;
- Antibacterial and bacteriostatic due to its high alkalinity - pH: 12.5;
 - High radiopacity: 7 mm of aluminum scale, according to the ISO 6876 standard;
 - Does not suffer setting contraction;
 - It can be used, in addition to conventional techniques, in single-cone techniques and thermoplastic filling.

USE TECHNIQUE

Use of Applicator Tips

1. Position the applicator tip in the syringe and perform the chemical decontamination of the assembly (tip + syringe);
2. We recommend using a disposable plastic cover on the syringe to avoid cross-contamination;
3. Confirm the material exits from the syringe before direct application to the channel;
4. Position the tip and syringe assembly properly into the canal and depress the plunger lightly to prevent excessive output of the product;
5. Before closing the syringe, retract the plunger to stop the outflow;
6. Clean any excess material on the tip of the syringe and close with appropriate pressure after use to prevent contact with moisture, which causes the product to dry;

7. Clean and disinfect the syringe in subsequent uses to avoid cross-contamination; in case of contamination with saliva or blood, discard it.

A. SEALING THE ROOT CANAL OF PERMANENT TEETH

Traditional technique - Lateral compression

1. Anesthetize, install absolute isolation and then perform the biomechanical preparation of the canal;
2. Remove the sealing tip and attach the applicator tip on the syringe;
3. Dry the canal only with absorbent paper cones without causing excessive dryness;
4. Apply BIO-C SEALER in the apical third of the canal directly with the syringe;
5. Insert the gutta percha cone in the canal according to the last file used covered with BIO-C SEALER;
6. X-ray for checking the correct filling of the canal;
7. Cut the cone at the desired height with heated instruments followed by vertical compression;
8. Remove, with water, excessive material from the canal walls, perform crown sealing and restoration.

Important:

- BIO-C SEALER does not suffer setting contraction;
- BIO-C SEALER can be used, in addition to conventional techniques, in single-cone techniques and thermoplastic filling;
- To use the capsule presentation, place it on the gun, depress the plunger to dispense the material on a glass plate or dispense it directly into the canal, performing one of the techniques mentioned above;
- Use conventional gutta removal techniques.

Single-cone technique

1. Anesthetize, install absolute isolation and then perform the biomechanical preparation of the canal;
2. Remove the sealing tip and attach the applicator tip on the syringe;
3. Dry the canal only with absorbent paper cones without causing excessive dryness;
4. Apply BIO-C SEALER in the apical third of the canal directly with the syringe;
5. Insert the gutta percha cone;
6. X-ray for checking the correct filling of the canal;
7. Cut the cone at the desired height with heated instruments followed by vertical compression;
8. Remove excessive material from the canal walls with suitable irrigating solutions for this purpose, perform crown sealing and restoration.

B. INTERNAL REABSORPTION TREATMENT

1. Anesthetize and install absolute isolation;

2. Remove granulation tissue from reabsorption area with sharp cures;
3. Neutralize the area with calcium hydroxide paste;
4. Remove the calcium hydroxide in the next session;
5. Dry the canal only with absorbent paper cones without causing excessive dryness;
6. Insert BIO-C SEALER with the applicator tip across the canal, prioritizing the reabsorption area;
7. Insert the accessory gutta percha cones according to the lateral compression technique;
8. Cut the cones at the upper part of the reabsorption with heated instruments. Perform vertical compression for better disposal of the sealer in the reabsorption area;
9. X-ray for checking the correct filling of the reabsorption and canal area;
10. Fill the remaining of the canal with gutta percha;
11. Perform crown sealing with glass ionomer or other material of your preference and restore the tooth;
12. X-ray and follow up for at least two years

Filling removal

- Use conventional gutta removal techniques.

WARNINGS

- Use absolute isolation in all indications;
- Do not use BIO-C SEALER to fill the canal without the gutta percha cones;
- The product is sensitive to moisture, so properly close the packaging with proper pressure to prevent dryness;
- Never store it in a refrigerator;
- Dry the canal with paper cones, but without excessively drying the canal; the moisture from the dentin tubules will initiate the material's setting reaction.
- Use the product only after controlling the infection, because at this stage the material's setting reaction does not occur due to the acidic pH in the area;
- Avoid causing excessive overflow of the product to the periapical region;
- Do not reuse the applicator tips;
- Single-use product;
- Clean and sterilize the syringe in subsequent uses to avoid cross-contamination.

PRECAUTIONS

- Wear safety goggles, mask and gloves when handling the product. In case of contact with eyes or skin, wash with water;

- Do not use the product after its expiration date;
- Do not use the product if its packaging is damaged.
- Do not use it in patients that report sensitivity to the components of the formula.
- Product not tested in pregnant and lactating women;
- Keep the product away from children.

STORAGE

- Keep the product in a dry and ventilated place between 15 and 30°C and with relative humidity below 60%.
- Do not store it in a refrigerator.
- Do not store the product near ammonia, ammonium nitrate and products containing chlorine. Avoid using disinfectant solutions that contain any of these ingredients.
- * Product used exclusively by the dental surgeon

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