

## **CONTACTO PAPEL - Bula**

### **PORTUGUÊS**

#### **INSTRUÇÕES DE USO:**

- Posicionamento do CONTACTO na superfície oclusal dos dentes ou próteses a serem testadas;
- Articulação dos dentes para marcação dos pontos de contato oclusais;
- Ajustes com instrumentos adequados onde se fizer necessário.

**Composição:** Papéis especiais, ceras vegetais, pigmentos, óleos vegetais, parafina e vaselina.

**Advertência:** Para obter melhores resultados, secar as superfícies dentais a serem testadas.

Conservar em lugar seco e arejado.

### **ESPAÑOL**

#### **INSTRUCCIONES DE USO:**

- Posicione CONTACTO sobre las superficies oclusales de dientes y restauraciones;
- Articular las arcadas superior e inferior para marcar los puntos de contactos oclusales;
- Haga los ajustes necesarios con instrumentos adecuados

**Composición:** Papeles especiales, ceras y aceites vegetales, pigmentos, parafina, vaselina.

**Atención:** Para mejores resultados, secar las superficies oclusales antes de marcar los puntos contactos. Almacenar en local seco y aireado.

### **ENGLISH**

#### **INSTRUCTIONS FOR USE:**

- Place CONTACTO on the occlusal surfaces of teeth or restorations to be tested.
- Articulate upper and lower arches to obtain the occlusal marks.
- If necessary, adjust the occlusal marks with adequate instruments.

**Composition:** Special paper, vegetal wax, vegetal oil, pigments, paraffin and petroleum jelly.

**Attention:** Dry the occlusal surfaces to obtain the best results.

Keep in dry and ventilated place.