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## PORTUGUÊS

### GENERALIDADES

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## ENGLISH

### GENERAL DESCRIPTION

**Clàriant D-Sense** is a desensitizing gel with a potassium nitrate and sodium fluoride base. Its formula acts on neural sensitivity mechanisms and on the sealing of dentinal tubules.

### DEFINITION

Desensitizing Potassium Nitrate and Sodium Fluoride based gel.

### INDICATIONS

It is indicated for the treatment of tooth sensitivity caused by the whitening processes, periodontal treatment, thermal stimuli and food.

### COMPOSITION

- Potassium Nitrate;
- Sodium Fluoride;
- Thickening agent;
- Glycol;
- Neutralizing agent;
- Preservatives;
- Deionized water.

### INSTRUCTIONS FOR USE

1. **Clàriant D-Sense** gel can be applied directly on the tooth or with the aid of a bleaching tray;
2. At the office, after tooth scaling: under relative isolation, apply a layer on each tooth;
3. This layer should be left for at least 10 minutes, keeping the patient's mouth during this period;
4. Before removal, the gel can be burnished with polishing brushes, rubber cups or felt discs at low speed;

- When using the bleaching tray, after cleaning the mouth, apply in the tray 1 drop of Clariant D-Sense gel on the area correspondent to the sensitive tooth;
- The tray should stay in the mouth for 30 minutes.

#### STORAGE

- Store the product at room temperature, between 5 and 25°C. Do not freeze the product. Do not expose to direct light.

#### PRECAUTIONS

- Clariant D-Sense must be completely removed from the mouth. The product should not be swallowed;
- Depending on patient sensitivity, more applications can be made, but never more than once per day;
- Clariant D-Sense must be used under the supervision of a dentist;
- The patient must be in good oral and general health before treatment using this product;
- This product cannot be used on patients who are allergic to any of its components.

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