ESPÁTULA

PORTUGUÊS

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ENGLISH

DEFINITION: Spatula for the sculpture of ceramic/porcelain and resins with ultra flexible blades and depth gauge (blade 1 with depth gauge: The distance between the mark on blade number 1 and its tip is 0.7 mm. Such reference is to check the thickness of the applied porcelain. A minimal thickness of 0.7 mm is recommended to achieve the best esthetic results.

COMPOSITION: Handle: aluminum/Blades: stainless steel.

INDICATION: Sculpture of ceramic/porcelain and resins.

WARNING: How to take care of the blades: **a.** Handle gently; although highly flexible, they may deform in case of falls. **b.** Do not use corrosive cleaning products; clean with water or alcohol and dry immediately. **c.** Store in a dry, moisture-free environment.

INSTRUCTIONS FOR USE: Adaptation of blade to handle: **1.** Open the cross-shaped connecting part of handle. **2.** Insert and fix blade. Selection of blade: **a.** blade 1 (spear-shaped): sculpture of ceramic and depth gauge. **b.** blade 2 (saw-shaped): sculpture of bridge connectors and details of facial and incisal areas of teeth. **c.** blade 3 (spoon-shaped): sculpture of lingual, occlusal and facial areas of teeth. Blades 4, 5 and 6: for resin sculpture. Sterilizable in autoclave and dry heat.