ÁCIDO FOSFÓRICO

PORTUGUÊS

DEFINIÇÃO

Ácido fosfórico 37%.

COMPOSIÇÃO

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37% phosphoric acid.

COMPOSITION

Orthophosphoric acid; water; thickener; pigments (used for conditioning of the dental surface).

INDICATIONS

Direct composite resin restorations and for the adhesive cementation of prefabricated posts (metal and fiber); cast metal posts and indirect restorations (metal, porcelain and composite resin); bonding of orthodontic brackets and amalgam.

WARNINGS

Keep the product in a dry place under shelter of the sun. During transportation and storage the temperature must be kept below 28°C. Do not use this product in patients with reported sensitivity to its components. Use gloves, mask and eye protection when handling the product.

TECHNIQUE OF USE

A. FOR PROCEDURE IN ENAMEL:

- Ensure that the conditions are suitable (isolation of operating field, perfect control of humidity);
- Apply the PHOSPHORIC ACID gel to the enamel for 15 to 30 seconds.
- Remove the PHOSPHORIC ACID gel with water and air jet for 15 seconds.
- Dry the enamel surface with air jet until complete removal of the humidity.

B. FOR PROCEDURE IN DENTIN:

- Ensure that the conditions are suitable (isolation of operating field, perfect control of humidity).
- Apply the PHOSPHORIC ACID gel to the dentin for 15 seconds;
- Remove the PHOSPHORIC ACID gel with water and air jet for 15 seconds;
- Dry the dentin surface in accordance with the technique to be used.

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