Instructions for use

0524082016

Choose your language

- | | |
|-----------------------|-----------------------|
| PT Português | SK Slovensky |
| EN English | PL Polski |
| ES Español | ET Eesti |
| FR Français | LT Lietuviškai |
| IT Italiano | LV Latviski |
| RO Română | CS Český |
| NL Nederlands | GA Gaeilge |
| DE Deutsch | HU Magyar |
| SV Svenska | TR Türkçe |
| DA Danish | MT Malti |
| FI Suomi | BG Български |
| HR Hrvatski | RU Русский |
| SL Slovenščina | EL Ελληνικά |

Português

DEFINIÇÃO

Ácido Fluorídrico 10%.

INDICAÇÕES

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10% Hydrofluoridric Acid.

INDICATIONS

Cementation of all-ceramic (porcelain) restorations.

Repair of porcelain with composite resin.

Etching of orthodontic brackets to be cemented in porcelain.

COMPOSITION

Hydrofluoridric acid, water, thickening agent and pigment.

TECHNIQUE FOR USE

A. CEMENTATION OF ALL-PORCELAIN RESTORATIONS

1. Isolate the external surface of the porcelain restoration with a thin layer of wax.
 2. Etch the internal surface with Angelus® Porcelain Etchant (10% Hydrofluoridric Acid) for 1 minute*. Wash abundantly with water and dry thoroughly.
- *Leucite-based porcelains are etched only for 20 (twenty) seconds.
3. Apply SILANE ANGELUS® on the etched surface for 1 minute and gently dry with air.
 4. Proceed with cementation using a resin cement, according to the manufacturer's instructions of the cement.

B. PORCELAIN REPAIR

1. Make a 2 to 3 mm bevel at the fractured area and wash with an air/water spray.
2. Etch the beveled area with Angelus® Porcelain Etchant (10% Hydrofluoridric Acid) for 1 minute. Wash abundantly with water and dry thoroughly.
3. Apply SILANE ANGELUS® for 1 minute and gently dry with air.
4. Apply bonding agent FUSION-DURALINK® ANGELUS® and a restorative composite resin, according to the manufacturer's instructions.

PRECAUTIONS

Hydrofluoridric acid is corrosive. Contact with enamel, dentin, eyes, skin and oral tissues must be avoided.

Use of eye protection, gloves and mask by dentist and assistant, and eye protection by patient is highly recommended.

When applied in the patient's mouth, oral tissues should be protected with vaseline and, whenever possible, use rubber dam isolation. In case of accidental contact with eyes, skin or oral tissues, wash abundantly with water. If necessary, seek medical attention.

Keep packaging closed after use.

Do not use it in patients who are sensitive to any of the components.

STORAGE

Store the product in a dry place protected from light.

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