## Instructions for use

0320102016

### Choose your language

- |                       |                       |
|-----------------------|-----------------------|
| <b>PT</b> Português   | <b>SK</b> Slovensky   |
| <b>EN</b> English     | <b>PL</b> Polski      |
| <b>ES</b> Español     | <b>ET</b> Eesti       |
| <b>FR</b> Français    | <b>LT</b> Lietuviškai |
| <b>IT</b> Italiano    | <b>LV</b> Latviski    |
| <b>RO</b> Română      | <b>CS</b> Český       |
| <b>NL</b> Nederlands  | <b>GA</b> Gaeilge     |
| <b>DE</b> Deutsch     | <b>HU</b> Magyar      |
| <b>SV</b> Svenska     | <b>TR</b> Türkçe      |
| <b>DA</b> Danish      | <b>MT</b> Malti       |
| <b>FI</b> Suomi       | <b>BG</b> Български   |
| <b>HR</b> Hrvatski    | <b>RU</b> Русский     |
| <b>SL</b> Slovenščina | <b>EL</b> Ελληνικά    |

# Português

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# English

Chemical Bonding agent for metal, Zirconia and Alumina

## INTRODUCTION

MZ Primer is a one-bottle solution that promotes the bonding of resins (composite, cement, acrylic) to a variety of substrates such as metals and ceramics, including zirconia and alumina. Simple and versatile, without the need of complicated procedures, it provides long-term, reliable results.

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In situations where it is necessary to increase and stabilize bonding between organic materials such as resins (composite, cement, acrylic) and inorganic materials such as metals and ceramic surfaces.

### A. CLINICAL USE:

- Cementing of zirconia prostheses;
- Cementing of metal ceramic prostheses;
- Cementing of precious and non precious alloy prostheses;
- Cementing of prefabricated metal posts and cast posts;
- Repairs of metal ceramic and zirconia prostheses.

### B. LABORATORY USE:

- Application of acrylic resin on metal areas of Removable Partial Prostheses;

- Repair of acrylic resin of Removable Partial Prostheses;
- Application of acrylic resin on metal areas of implant protocols.

## COMPOSICIÓN

pHEMA, methacrylate, PMDM, benzoyl peroxide and solvent (acetone).

## INSTRUCTIONS FOR USE

### A. CLINICAL USE

Cementing of metal-free zirconia prostheses

- Preparation of the retainer tooth:

1. On natural tooth surfaces, use a bonding agent (e.g. Fusion Duralink) according to its specific guidelines;
2. If the retainer tooth presents metal surfaces (e.g. a cast metal post), these should be roughened with an abrasive microetcher/sandblaster or diamond bur, washed with abundant water and then receive a layer of MZ PRIMER (wait for 3 minutes).

**IMPORTANT:** Do not allow MZ PRIMER to contact the oral tissues.

- Preparation of the zirconia restoration inner surface:

1. Roughen the inner areas of the restoration with an abrasive microetcher/sandblaster (with alumina particles);

**IMPORTANT:** Follow the ceramic/porcelain manufacturer's instructions regarding the adequate time and pressure of sandblasting.

2. Clean the restoration in an ultrasound unit for 1 minute, immersed in alcohol; Do not use phosphoric acid for cleaning after micro etching.
3. Wash the area with abundant water and dry with air;
4. Apply a uniform layer of MZ PRIMER on the surface, wait for 3 minutes and dry with air;
5. Apply a layer of the bonding agent (e.g. Fusion Duralink) following its specific guidelines;
6. Apply the resin cement to the restoration (e.g. Cement Post) and place it on the retainer tooth. Follow the cement manufacturer's guidelines regarding curing.

Cementing of metal ceramic and metal prostheses

- Preparation of the retainer tooth:

1. On natural tooth surfaces, use a bonding agent (e.g. Fusion Duralink) according to its specific guidelines;
2. If the retainer tooth presents metal surfaces (e.g. a cast metal post), these should be roughened with an abrasive microetcher/sandblaster or diamond bur, washed with abundant water and receive a layer of MZ PRIMER (wait for 3 minutes).

**IMPORTANT:** Do not allow MZ PRIMER to contact the oral tissues.

- Preparation of internal surfaces of the metal restoration:

1. Sandblast (with aluminum oxide) the inner areas of the metal coping until it becomes opaque and homogeneous;

2. Clean the restoration in an ultrasound unit for 1 minute, immersed in water. Do not use phosphoric acid for cleaning after micro etching.

3. Wash the area with abundant water and dry with air;

4. Apply a uniform layer of MZ PRIMER on the inner surface, wait for 3 minutes and dry with air;

5. Apply a layer of the bonding agent (e.g. Fusion Duralink) according to its specific guidelines;

6. Apply the resin cement to the restoration (e.g. Cement Post) and place it on the retainer tooth. Follow the cement manufacturer's guidelines regarding curing.

Cementing of prefabricated metal and cast posts

Follow the same procedures described for the cementing of metal ceramic prostheses on natural tooth surfaces.

Repair of metal ceramic prostheses with composite resin

**IMPORTANT:** For this procedure, always isolate with rubber dam to avoid any contact between the chemicals and the oral tissues.

1. With a diamond bur, bevel the ceramic repair site; if there is exposed metal, roughen it with a sandblaster or diamond bur;
2. Apply ANGELUS PORCELAIN CONDITIONER (10% Hydrofluoric Acid) on the beveled surface, wait for 1 minute, wash with water and dry with air;

**IMPORTANT:** Protect the oral tissues from the corrosive action of the hydrofluoric acid.

3. Apply ANGELUS SILANO on the ceramic surface, wait for 1 minute and dry with air;
4. Apply a uniform layer of MZ PRIMER on the metal, wait for 3 minutes and dry with air;
5. Cover the metal with ANGELUS OPAK opaque resin;
6. Apply FUSION DURALINK bonding agent according to its instructions;
7. Repair the fractured site with a composite resin.

Repair of zirconia prostheses

**IMPORTANT:** For this procedure, always isolate with rubber dam to avoid any contact between the chemicals and the oral tissues.

1. Bevel the fractured restoration site with a diamond bur;
2. Apply MZ-PRIMER, wait for 3 minutes and dry with air;
3. Apply FUSION DURALINK bonding agent according to its instructions;
4. Repair the fractured site with a composite resin.

### B. LABORATORY USE (SELF-CURED AND HEAT-CURED ACRYLIC RESIN)

Manufacture and repair of removable partial prostheses with acrylic resin

Manufacture of implant protocols with acrylic resin

1. Using aluminum oxide, sandblast the metal surface that will receive the acrylic resin;
2. Clean the framework with an ultrasound unit for 1 minute, immersed in water or wash abundantly with

water and dry with air;

3. Apply a uniform layer of MZ PRIMER on the site, wait for 3 minutes and dry with air;

4. If necessary, use an opacifying agent to mask the metal color (e.g. Angelus PINK OPAK);

5. Apply the acrylic resin and follow its instructions regarding polymerization.

#### WARNINGS

- This product is contraindicated in cases of allergy to any of its components and should not be used in situations not listed in these instructions;

- Systemic side-effects are not known, however in rare cases, some components of MZ PRIMER may lead to patient sensitization, therefore do not use in these situations;

- This product may irritate skin and oral tissues. If such contact occurs, wash immediately with abundant water;

- Commercial gloves do not provide long term protection against sensitization effects caused by methacrylates.

#### PRECAUTIONS

- Keep this product away from heat and fire sources due to its extreme flammability;

- Discard brushes or applicators used with the product;

- Avoid saliva, blood or water contamination on MZ PRIMER application sites. If this occurs, the procedure must be totally restarted;

- Do not contaminate the MZ PRIMER solution by replacing the brush or applicator inside the bottle;

- Protect the product from light and heat;

- Close the MZ PRIMER bottle tightly to avoid evaporation of the components, which could alter the efficacy of the product.

#### STORAGE

- After the initial opening of the bottle, it is recommended that the product is stored in temperatures between 2°C and 8°C; this is especially important in warm climates.

- Validity: 24 months. Do not use after the expiry date.

**KEEP AWAY FROM CHILDREN. FOR DENTAL USE ONLY.**

This product was fabricated for dental applications only and must be used according to its instruction manual. The manufacturer is not responsible for any harm caused by uses not described in the instructions or by inappropriate handling. The user must check, under their responsibility and before use, if the product is compatible with the desired application, especially when such application is not indicated as described in the instructions.

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