EVIDENCIADOR DE PLACA

PORTUGUÊS

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ENGLISH

DESCRIPTION: Disclosing solution with two chromatic indicators for coloring old plaque (blue) and new plaque (red).

COMPOSITION: Erythrosine Sodium, Bright Blue Pigment, Methylparaben, Propylparaben, filtered water.

WARNINGS AND PRECAUTIONS: 1 - Always keep the bottle firmly closed, in a cool place protected from direct sunlight. 2. Keep out of the reach

of children. 3 - If there is accidental contact with the eyes, wash them abundantly with water. 4 - Wear personal protection appliances (mask, goggles and gloves). 5 - Do not swallow: if swallowed by accident, seek medical aid with the instructions at hand.

STORAGE: Keep it in a dry and airy place out of the light and reach of children.

TECHNIQUE FOR USE: 1 - Place 4 to 5 drops in the mouth, ask the patient to spread them on the teeth with the tongue, and leave to act for 10 seconds. 2 - Ask the patient to rinse the mouth until the water coming out is no longer stained. All the dental surfaces where there is old bacterial plaque will be stained blue and where there is recent plaque, red. 3 - Instruct the patient regarding the perfect technique for brushing, for eliminating plaque. 4 - Apply the Plaque Disclosing Agent again; if the brushing has been perfect, there will not be any colored dental surface.

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