3D SAFEMIX

PORTUGUÊS

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PARA USO COM CARTUCHO DE SILICONE

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TECHNIQUE FOR USE

FOR USE WITH SILICONE CARTRIDGE

1. Insert the posterior part of the cartridge in the anterior part of the gun. Fix the cartridge with the gun lock.
2. Untwist the cartridge cap and throw it away.
3. Press the gun trigger to engage the gun plunger in the cartridge.
4. Before adapting the automix tip in the cartridge, press the trigger slowly and allow a 0.5 cm length of the impression material to leak. Both pastes – catalyst and base – must flow freely. Clean the tip of the cartridge.
5. Adapt the automix tip in the cartridge and rotate it one quarter turn to lock it in the correct position.
6. Press the trigger smoothly to mix and dispense the material. When the operator stops pressing the trigger, the material flow is interrupted.
7. Leave the automix tip as a cartridge cap until the next impression, when this used tip will be replaced by a new one. To proceed with a new impression, follow steps 4 through 7.

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ATTENTION: This product must be used according to the instructions described in this manual. The manufacturer is not responsible for failure or damage caused by incorrect handling or use.