# SYRINGE SLEEVE

angie  
by angelus®

## Instructions for use

Choose your language

PORTUGUÊS

ENGLISH

ESPAÑOL

FRANÇAIS

ITALIANO

ROMÂNĂ

DUTCH

DEUTSCH

SVENSKA

DANISH

SUOMI

HRVATSKI

SLOVENŠČINA

SLOVENSKY

POLSKI

EESTI

LIETUVIŠKAI

ČESKY

MAGYAR

TÜRKÇE

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### Simbologia | Symbology | Simbología | Semboller

	Cuidado, consultar documentos acompañantes. For care, see accompanying documents. Cuidado, consultar documentos adjuntos. Ek belgelere bakınız.
	Manter ao abrigo do sol. Keep under shelter of the sun. Mantener al abrigo del sol. Işıktan koruyunuz.
	Manter seco. Keep dry. Mantener seco. Kuru yerde saklayınız.
	Produto de uso único. Disposable product. Producto de uso único. Tek kullanımlık ürün.

# PORTUGUÊS

Luva para Carpule – Jacarezinho

## **INDICAÇÃO**

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Syringe Sleeve – Alligator

## **INDICATION**

Diminishing of anxiety caused by needle fear during anesthesia.

## **COMPOSITION**

Elastomer and polypropylene.

## **INSTRUCTIONS FOR USE**

1. Insert the syringe prepared with cartridge and needle in the sleeve.
2. Place into the area of the mouth to receive anesthesia.
3. Press the rear feet of the Alligator Sleeve with the index and middle fingers to expose the needle. Proceed with the injection of the anesthetic solution. After completing the injection, release the rear feet of the sleeve (which will hide the needle again) and remove the assembly from the patient's mouth.

## **PRECAUTIONS/WARNINGS**

1. Before using, proceed the sterilization in autoclave;
2. The lower part of Alligator Sleeve must be positioned towards the dentist, enabling visual verification of the flow of the anesthetic solution and any potential bleeding;
3. Place the syringe inside the sleeve, ensuring that the needle does not come into contact with the internal wall of the product;
4. Place the fingers on the rear feet of alligator and never on the plunger. This is the most common error when using the product for the first time;
5. The

use of the adhesive stickers by children under the age of 7 must be monitored by an adult. 6. In case of donation of the sleeve, disinfect it.

Note: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority health in which the user and/or patient is established.

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